

# One Step Gonorrhea Swab Test

**Gonorrhea** is one of the common sexually transmitted diseases (STD) in the whole world. Incidence of a disease is increasing year after year. It is the key approach to early and fast diagnosis for early treatment and shutting infection off.

**Wondfo One Step Gonorrhea Swab Test** is a rapid test for the visual detection of Gonorrhea antigen, in the secretory specimen from urogenital system, as an aid in the diagnosis of gonococcus infection.



## SPECIMEN

Female endocervical swab sample  
Male penile swab sample

## DETECTION TARGET

Gonorrhea antigen

## DETECTION TIME

10-15 mins


## SENSITIVITY

95.04%

## SPECIFICITY

98.66%

## TEST PROCEDURE

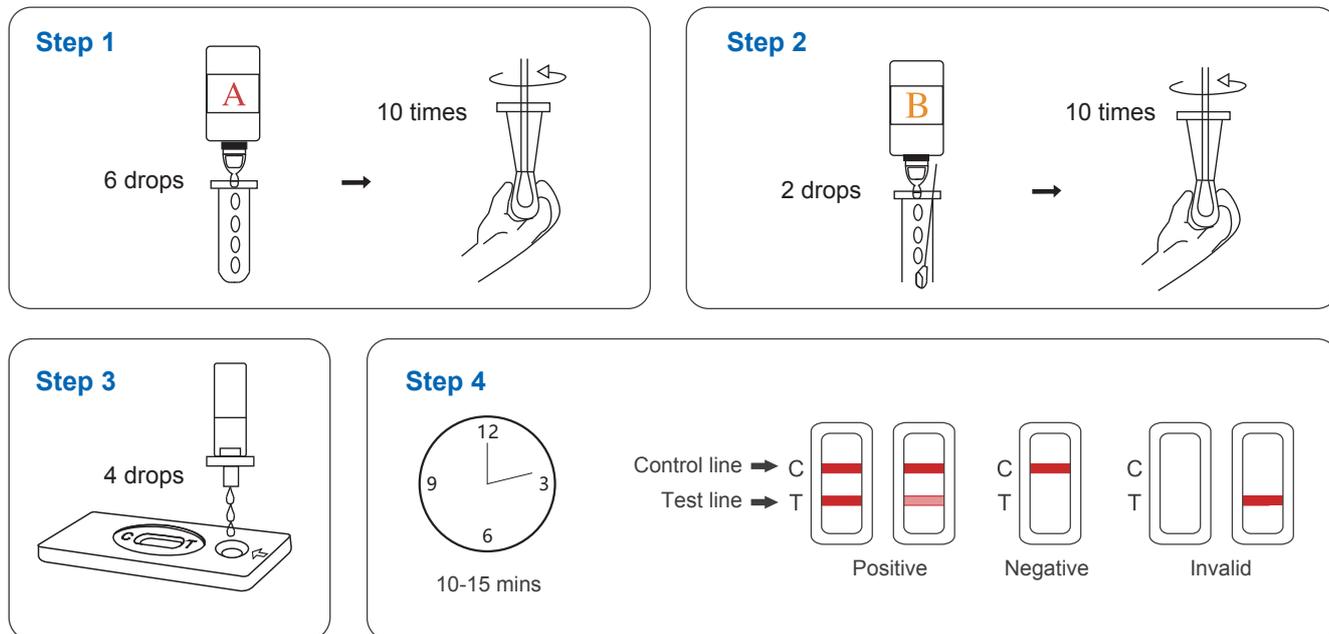

## PERFORMANCE CHARACTERISTICS

Wondfo kit	Commercial kit		Total
	Positive	Negative	
Positive	364	9	373
Negative	19	664	683
<b>Total</b>	<b>383</b>	<b>673</b>	<b>1056</b>

**Sensitivity of Wondfo One Step Gonorrhea Swab Test: 95.04%**

**Specificity of Wondfo One Step Gonorrhea Swab Test: 98.66%**

## ORDER INFORMATION

Item Code	Description	Pack Size	Qualification
W36-C	One Step Gonorrhea Swab Test	25 Tests/kit	CE
W036P0010	One Step Gonorrhea Swab Test	40 Tests/kit	CE

